

arthronet® instruments

They are developed and designed for the remove of tissue in joint interiors.

The form and surface design is given an optimal leadership at the joint cartilage. The instruments are equipped with a rupture joint in the handle, which prevents a break in case of an extreme over-load in the invasive area. They are sterilizing with the help of the known sterilization methods.

arthronet®-instruments are only to use if:

1. The function of the complete instrument is given.
2. The seize or cutting performance is still sufficient.
3. There isn't a mechanical damage.
4. A suitable cleaning and sterilization method was used.

Warning advice!

arthronet®-instruments isn't to use and to replace immediately if:

1. There is a mechanical defect.
2. The grip or cutting performance is not sufficient.
3. If no sterilization or cleaning before using is possible.

In case of damage while using the instrument, it has to be replaced immediately.

Basicly, arthronet® instruments only shall be used and sterilized by technically qualified staff!

Possible adverse consequences

- Infections because of unproper sterilization.
- Complications from using a mechanically defect instrument.

Responsible for the selection of the sterilization method and with that for the sterility of the treated products is the runner of the sterilization system.

Identification: The instruments are sign with a serial number.

arthronet® **Germany Ltd. & Co. KG**

Geilenbacher Straße 31

D-51399 Burscheid

Tel. 02174-7655-0

Fax 02174-7655-55

e-Mail: service@arthronet.de

