

## arthronet® Suction Punch

**It is developed and designed for the remove of tissue in joint interiors.**

The form is ideally usable causing in the front horn area. The blade can be turned round 360° depending on need in 45° steps. It is analyzable and with the help of the known sterilization methods sterilizing. The Suction Punch can be used with blades in three different diameters (3.5 - 4.5 - 5.5 mm).

**The arthronet®-Suction Punch only has to be used if:**

1. The function of the complete instrument is given.
2. The cutting performance is still sufficient.
3. There isn't a mechanical damage
4. A suitable cleaning and sterilization method was used.

### **Warning advice!**

**The arthronet® Suction Punch isn't to use and to replace immediately if:**

1. There is a mechanical damage.
2. The cutting performance is not sufficient.
3. No suitable cleaning and sterilization can be first of all carried out

**In case of damage while using the Suction Punch, it has to be replaced immediately**

**Basically, the arthronet® Suction Punch only shall be used and sterilized by technically qualified staff !**

### **Possible adverse consequences**

- Infections because of improper sterilization.
- Complications from using a mechanically defect Suction Punch.

**Responsible for the selection of the sterilization method and with that for the sterility of the treated products is the runner of the sterilization system**

**Identification: The Suction Punch is labeled with a serial number. The blades carry a lot- number and the last four places of the article- number.**

arthronet® Germany Ltd. & Co. KG

Geilenbacher Straße 31

D-51399 Burscheid

Tel: (\*\*49) - 02174-76550

Fax: (\*\*49) - 02174-7655-55

e-Mail:service@arthronet.de

